

PRODUCT MANUAL

Product Name: Particulate Respirator

Legal Manufacturer: Toscana Servizi Srl

Model: LN-M01

Regulation: PPE (EU) 2016/425
PPE-R/02.075 version 2
“Type-examination by notified body FORCE Certification A/S, 0200, DK-2605”



EU DECLARATION OF CONFORMITY is available at the following link
<https://www.anditoscana.it/area-download/>

According to Commission Recommendation (EU) 2020/403 of 13 March 2020 paragraph 3 and 4 and actual WHO recommendations, FFP2 or equivalent filtering half-masks can be used for protection against SARS-CoV-2.

Application:

This filtering half mask is manufactured for COVID-19 protection only. As requested by World Health Organization recommendations, for this specific use, the nominal protection factor given by this filtering half mask is the same than the FFP2 nominal protection factor defined in EN 149:2001+A1:2009. This filtering half mask is not a filtering half mask for general use and shall not be used for purposes other than protection against COVID-19.”

Product composition:

PP nonwoven fabric, non-latex cotton, melt-blown fabric, polytene, polyurethane fiber

Attention before use:

Please read these instructions thoroughly before use. Our face mask is non-re-usable particle filtering half mask. This mask is intended exclusively for the personal protection, it is not medical face mask. Do not use the masks in areas with oxygen deficiency, do not use the mask in areas with pollution aerosols or particles.

Warning:

Don't use in areas with open flames
The correctly fitting is crucial for the performance of the mask

How to Use:

1. Apply the respirator on face and hang the ear loops on ears.
2. Press nose clip and make the respirator fixed on nose.
3. Adjust the mask and nose clip to make the respirator cling to your face and nose tightly.

Instructions to remove and discard the mask:

1. To remove the mask, use thumbs and index fingers to remove both ear straps
2. Carefully dispose the used mask adequately
3. Wash or disinfect hands

Cautions and Warning:

1. Please check the integrity and validity of the package before use. If the package is damaged, don't use it;
2. Prior to use, pls. check fit of particle filtering half mask.
3. It is unlikely that the requirements for leakage will be achieved if facial hair passes under the face.
4. Pls. note air quality (contaminants, oxygen deficiency).
5. Do not use it in explosive atmosphere.
6. Pls. note that the particle filtering half mask shall not be used for more than one shift.
7. This product is disposable. Please treat it according to the local environmental protection requirements after use, and do not discard it at will.

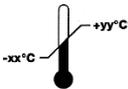
Storage:

1. Keep in cool place. Avoid sunlight and heat source. Avoid press or damage of box.
2. Keep in place with humidity less than 80%, no corrosive gas and well ventilated clean room.

Specifications: 10.5cm*15.5cm

Shelf Life: two years after production date

Symbols:

				
Store indoors, avoid direct sunlight	See information supplied by the manufacturer	Temperature range of storage conditions	Maximum relative humidity of storage condition	Symbol for CE Mark. This symbol certifies that a product has met European Union consumer safety, health, or environmental requirements.

Legal Manufacturer:

Name: Toscana Servizi Srl
Add: Via Caduti di Cefalonia, 96
50127 Firenze
ITALY
Tel/Fax: +39.055.4221748
Website: <https://www.anditoscana.it/>
Email: toscana@andi.it
stefanomirenghi@gmail.com

Place of Production:

Name: Guangzhou Renault Biotechnology Co., Ltd.
Add: Room604, North Tower, R&F Ying Li Plaza, No.3-2,
Huaqiang Road, Tianhe, Guangzhou, China
【Manufacturer Add.】 No.1 Office Building, No.2 workshop,
No. 3workshop, No. 4workshop, No.6, Xinghong Street,
PinShanErChun Industrial Part, Shibi Subdistrict Panyu,
Guangzhou, China

Zip Code: 510000
Tel: +86 020-39293697
Fax: +86 020-39293697
Website: www.renaultbio.com